

SETUP GUIDE

uProbe

1. If you are using an IOS device please visit the App Store and search for “wirelessusg”. If you are using Android please send a Whats App with the subject “Android” to 072 9231431 and we will send you the download link.
2. Switch uProbe on by pressing the button on the device once.
3. Enable Wifi on your smartphone or tablet and choose uProbe, starting with either SL or SN
4. Use the serial number on the back bottom of the device, starting after the colon. All lower case.
5. Once the device is paired the antenna next to the battery symbol starts blinking faster.
6. Open the wirelessusg app and unfreeze uProbe by pressing the button on the device once.
7. Start scanning

Please make sure that your brightness is set to high on your smartphone or tablet, for best image results. Always make sure to wear the black wristband. If you need further assistance please send a whats app to 072 9231431 and we gladly assist you.